



TrioDocs

Version: 0.2.1

Date: May 08, 2025

Download the latest version on:
<https://docs.diy-trio.org>


Live Activity

Live Activity

Enable Live Activity

Default: *OFF*

With Live Activities enabled, Trio displays your current data on your phone's Lock Screen and the Dynamic Island. Once this is enabled, you will have the option to choose either a "Simple" Display or "Detailed" Display with the setting [Live Screen Widget Style](#).

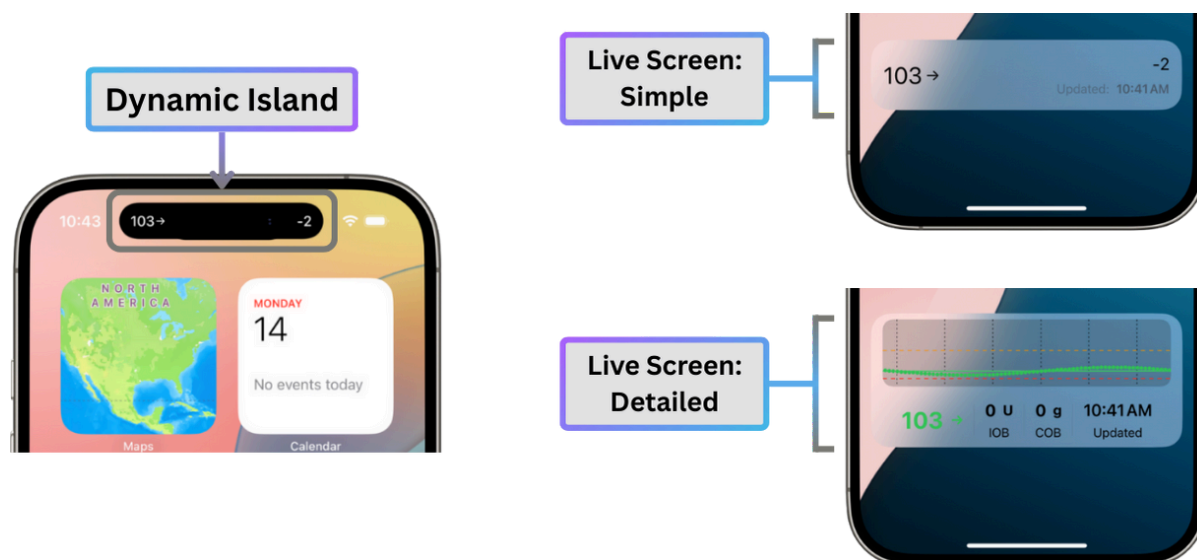

Live Screen Widget Style

Default: *Simple*

Customize your Live Activity Widget by choosing between "Simple" or "Detailed"

You'll find more details on both "Simple" and "Detailed" styles below:

Simple

Trio's simple widget displays your current glucose reading, trend arrow, delta, and the timestamp of the current reading.

